### Easy to maintain

#### Cassette LM 810001

LM extraction instruments are supplied in a convenient cassette that protects both the instrument and the handler during the maintenance cycle. The cassette keeps the instrument from puncturing the sterilization pouch, so sterility is more likely to be assured. The cassette can be color-coded.



#### Sharpening stone LM 818002

The blades, hand-finished and made from LM-DuraGradeMAX supersteel, will stay sharp and last a long time when properly used and maintained. Sharpen the extracting instruments occasionally using this round sharpening stone as a simple and quick way to keep them in top condition and always ready to use.

#### **LM**Dental

LM-Instruments Oy PL 88 (Norrbyn rantatie 8) FI-21601 Parainen, Finland Tel. +358 2 4546 400 Fax +358 2 4546 444 info@lm-dental.com www.lm-dental.com

## Root tip instruments

#### Root Tip Pick LM 769-771 XSI

- for the removal of root tip
- suits well in all difficult to reach areas
- available only with LM-ErgoMax (XSI) handle



### LM-RootOut LM 812210

- designed for delicate root teasing procedures
- not recommended for more substantial root tip removal
- similar to Heidbrink root tip picks
- ergonomically designed handle for a good grip



Contact your local distributor





# **LM**SlimLift

# Luxating instruments designed for extremely narrow places

The LM-SlimLift instruments are especially designed to reach places you cannot go with traditional luxating instruments. It allows tooth extraction to be done as atraumatically as possible. This is important in order to enable rapid healing. The extremely fine tip of the instrument is introduced into the very narrow periodontal space. The instrument is slowly advanced towards the apex of the root while turning the instrument gently back and forth. The LM-SlimLift is not suitable for use as an elevator.



- Ergonomic design comfortable to hold and easy to rotate
- Secure LM-ErgoTouch surface light, non-slip grip
- Well-balanced controlled grip
- Lightweight less strain on the hands
- Clear color-coding easy identification
- LM-DuraGradeMAX supersteel strong and sharp blades
- Tight seam at blade juncture hygienic



#### Angulations and sizes

